

# EQUIBIOME TEST

## Timeline



1

### ORDER TEST KIT ONLINE


Order online in the EquiBiome shop.  
Choose the International Kit if you live outside of the UK. The International Kits are compliant with the IATA and the UN.



2

### WHEN THE KIT ARRIVES

Collect the sample from your horse.  
Make sure it is fresh  
Make sure it is uncontaminated with straw/shavings/grass.

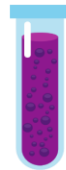

3

### SAMPLE SIZE

Only a small amount is needed, less is better than more, the sample must be covered completely by the liquid.



4

### A STAMP IS NEEDED

Please place a large letter stamp on the envelope.  
Please send track and sign if you want to be absolutely sure of delivery.



5

### LAB RECEIVES SAMPLE

As soon as we receive your sample we'll send you an email. The sample then goes to the lab. Once the analysis is complete the results will be emailed back to you.



6

### ONLINE

Once you have received your report please join the facebook group Equibiome Lounge, here you will find friendly support from others who have had the test done.